**FFP2 Faltnaske ohne Ventil /  
FFP2 face mask foldable without valve**  
Einweg Atemschutzmaske /  
Disposable, protective respirator mask

## Produktdaten / Description

REF	Farbe / Color	Packung / Package	Karton / Carton
ZH3161	weiß / white	40 Stück / 40 pcs. 135 x 135 x 210 mm	20 Packungen / 20 boxes 690 x 285 x 440 mm

## Allgemeine Informationen / General information

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**Important Note:** Non-observance of the information, especially in view of use, compatibility and durability may lead to damage to persons and / or material. Meditrade does not take any liability in case of improper use. This is a single use product – changes in material / infection risk cannot be excluded in case of multiple use. All information included correspond with the last document amendment date. Errors, literal mistakes and changes expected. In case of doubt please seek expert advice. Please find any up-dated product information on our homepage [www.meditrade.de](http://www.meditrade.de)

## Instructions for Use

1. Open the folded protective respirator mask with the nosepiece up and hold the protective respirator mask in hand, Allow headbands to hang freely

2. Position the protective respirator mask under the chin covering mouth and nose

3. Pull the lower headband over the head and position below ears. Pull the top headband on the back of head above ears for best fit

4. Press soft nosepiece to conform around the nose.

5. To check fit, cup both hands over the mask and exhale vigorously. If air flows around nose, tighten the nosepiece. If air leaks around the edge, reposition and adjust the headband for better fit.

6. Retest the seal and repeat the procedure until the protective respirator mask is sealed properly.

## Warning

1. Please check the mask for visible damage, dirt and the expiry date before use.
2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of protective respirator masks and result in sickness or death.
3. A properly selected protective respirator mask is essential to protect your health. Before using this protective respirator mask, consult an industrial hygienist or occupational safety professional to determine the suitability for your intended use. Always check the protective respirator mask for fit prior to use.
4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this protective respirator mask when oxygen concentration is less than 19.5%.
5. Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this protective respirator mask in explosive environment. Leave work area immediately and return to fresh air if, a) breathing becomes difficult, or b) dizziness or other distress occurs;
6. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this protective respirator mask.
7. Never alter or modify this protective respirator mask in any way.
8. Do not use for more than one shift (8 hours). No maintenance necessary.
9. Discard the used protective respirator mask after single use.
10. Keep protective respirator mask in the packaging box away from direct sunlight until use.

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